

iScreen™ Oral



Oral Fluid Drug Testing

Corporate Drug Testing Made Simple.

- No contact with specimen
- Simple saliva collection
- Split specimen well
- Detects six drugs
- Fast results

Brands You Can Trust™



iScreen™ Oral

Oral Fluid Drug Testing Corporate Drug Testing Made Simple.

Detect the presence of drugs in minutes.
It's drug testing efficiency you can count on.



iScreen™ Oral 6-panel
shown as a negative result.

Benefits to Oral Fluid Drug Testing:


- On-site testing
- Saves time & money
- Donor friendly
- Tests multiple drugs at once
- Simple one step procedure
- No contact with specimen

Drug Name:	Abbr:	Cutoff:
Amphetamine.....	AMP.....	50ng/mL
Methamphetamine.....	mAMP.....	50ng/mL
Marijuana.....	THC.....	100ng/mL
Phencyclidine.....	PCP.....	10ng/mL
Cocaine.....	COC.....	20ng/mL
Opiates.....	OPI.....	40ng/mL

Configuration:

6-panel.....I-DOA-765-2.....COC, THC, OPI, mAMP, AMP, PCP

Instructions

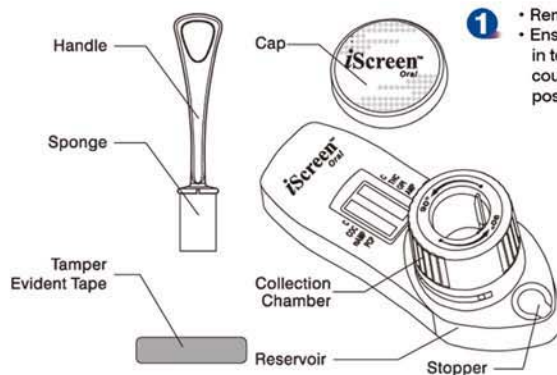


- 1**
- Remove cap.
 - Ensure reservoir is locked in testing position by turning counterclockwise as far as possible.


- 2**
- Insert sponge end of collector into mouth.
 - Actively swab the inside of the mouth and tongue with sponge. As soon as the sponge softens slightly, gently press the sponge between the tongue and teeth to ensure complete saturation.
 - The sponge is saturated when no hard spots can be detected. Collect for a total of (3) minutes before removing sponge.




- 3**
- Insert collector into device as shown.
 - Push collector down into chamber.
 - Turn collector clockwise to lock into chamber.

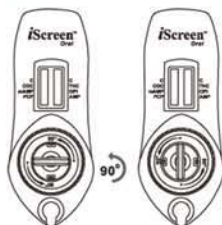


- 4**
- Wait **9 minutes**, remove collector from device and read results.

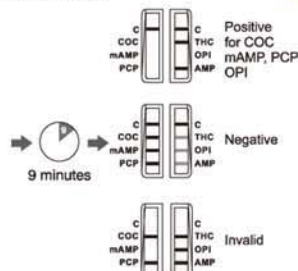

- 5**
- Secure cap back onto collection chamber.
 - Turn reservoir clockwise as far as possible to lock.
 - If results are positive, secure reservoir with tamper evident tape and send to laboratory for confirmation.

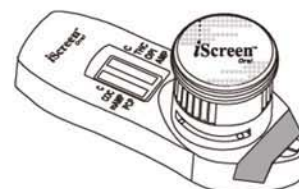


Instructions

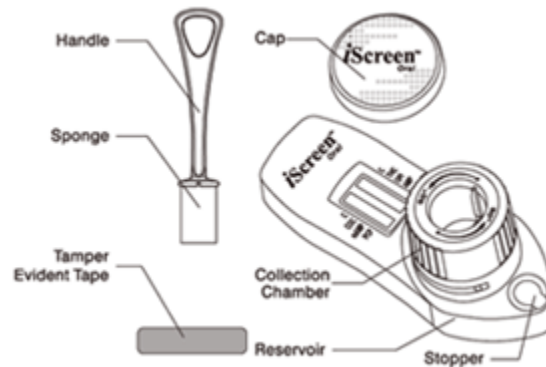


1. • Remove cap.
• Ensure reservoir is locked in testing position by turning counterclockwise as far as possible.




2. • Insert sponge end of collector into mouth.
• Actively swab the inside of the mouth and tongue with sponge. As soon as the sponge softens slightly, gently press sponge between the tongue and teeth to ensure complete saturation.
• The sponge is saturated when no hard spots can be detected. Collect for a total of (3) minutes before removing spor

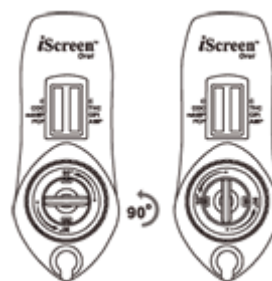

3. • Insert collector into device as shown.
• Push collector down into chamber.
• Turn collector clockwise to lock into chamber.

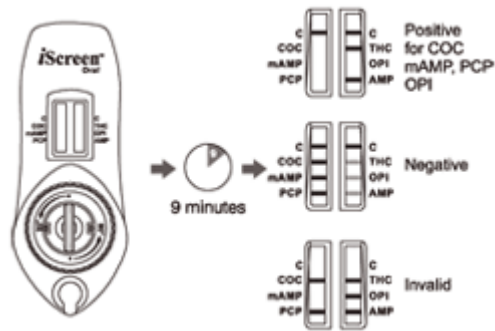

- 4 • Wait 9 minutes, remove collector from device and read results.

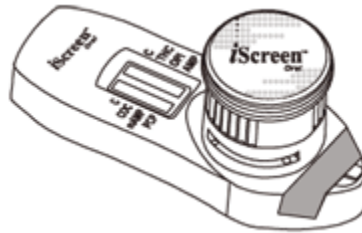

- 5 • Secure cap back onto collection chamber.
 • Turn reservoir clockwise as far as possible to lock.
 • If results are positive, secure reservoir with tamper evident tape and send to laboratory for confirmation