

QuikScreen®

One Step Onsite Drug Cup

Available with Adulteration Test

Unlike other onsite tests, QuikScreen® does not require any handling or manipulation to activate the test. Just dispense urine into the cup and read the results. The cup does all the work, IN ONLY 5 MINUTES.



- ★ SIMPLE PROCEDURE
- ★ EASY TO INTERPRET COLOR BANDS
- ★ BUILT-IN TEST CONTROL
- ★ ROOM TEMPERATURE STORAGE
- ★ 18 MONTHS SHELF LIFE
- ★ NO INSTRUMENTS NEEDED
- ★ OPTIONAL ADULTERATION STRIPS BUILT-IN!

QuikScreen® is the easy to use urinalysis drug test that detects drugs and drug metabolites at **SAMHSA** cutoff levels.

ORDERING INFORMATION:

	REGULAR CUP	ADULTERATION CUP
6 Test Cup	606XX	656XX
5 Test Cup	605XX	655XX
4 Test Cup	604XX	654XX

"X" denotes number representation of each drug combination.
Refer to catalog listing for combination and catalog number.

Create your own combination!

Tests available for MET, AMP, MAD, BZD, OPI, THC, COC, BAR, and PCP.

Cups available in five, six, seven, eight, and nine test formats.


QuikScreen® One Step Onsite Drug Cup

- ★ **Accuracy:** FDA 510(K) registered detection strips have the same high degree of accuracy when compared to SAMHSA certified laboratory screening methods.
- ★ **Self-Contained:** Each cup includes a temperature strip and identification label. Each cup is sealed in its own bag and has an 18-month shelf life.
- ★ **No Leakage:** The cup does not leak so a positive specimen can be shipped in the same container to a laboratory for GC/MS confirmation.
- ★ **Cost:** In addition to providing immediate results, QuikScreen® device costs less than half the price of laboratory testing.



Test procedure


1. Urinate into the cup, filling the cup with urine above the minimum level line.
2. Wait 5 minutes.
3. Read the results.
4. Refer to product insert for detailed instructions.

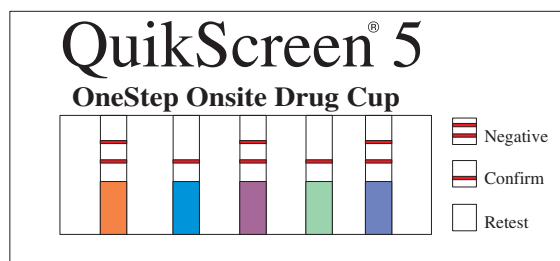

Confirm: One line indicates a positive result.

Negative: Two lines indicate a negative result.

Easy to read results

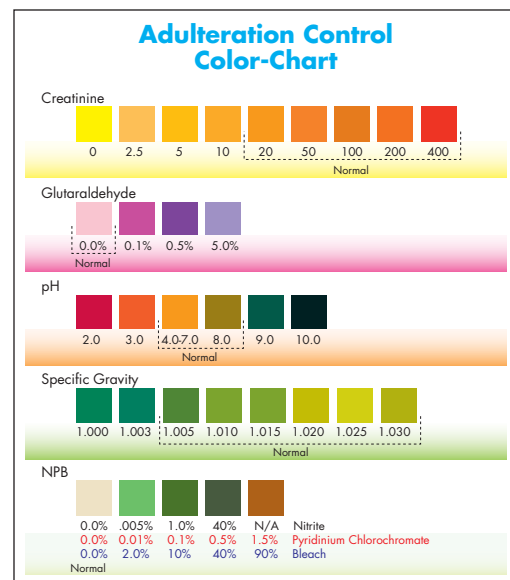

Compare adulteration results to color chart for devices with built-in adulteration check.

(Above chart is for illustration purpose only and should not be used to compare results.)

Contact DrugTesting-kits.com for more information about our QuikScreen® products. Call, fax, or e-mail us your order at sales@drugtesting-its.com. Visit us at www.drugtesting-its.com

Doc. ID: PDS-057
Revision: 01