



Catalog Number 31150S

PRODUCT INSTRUCTIONS

Q.E.D.® A150

Ethanol Control

Intended Use

OraSure Technologies Q.E.D.® A150 Ethanol Controls are intended to be used as a performance control in the determination of alcohol in saliva.

Summary and Principle

The use of quality control materials to monitor assay performance and reliability has been well established.

Contents

Catalog Number 31150S: 2 x 5 mL vials

Each vial contains an aqueous solution of ethanol with preservatives.

Precautions

For *in vitro* diagnostic use.

Reagent Preparation and Storage

OraSure Technologies Q.E.D.® A150 Ethanol Control is ready to use. No additional preparation is required.

Store at room temperature 15-30°C (59-86°F).

Unopened controls are stable until the date printed on the vial label. Replace the cap tightly immediately after use.

Procedure

OraSure Technologies recommends that you use controls on each new lot of Q.E.D.[®] tests. Substitute the control for saliva sample. Compare the Q.E.D.[®] test result to the Expected Range Below:

EXPECTED RANGE

CONSTITUENT	EXPECTED RANGE
Ethanol	0.075 - 0.130%
	75 - 130 mg/dL

Interpreting Test Results

The expected range is intended as a guideline to assess assay performance and reliability. Variations in user technique, reagent lot numbers and other systematic and random errors can affect the results you achieve. Contact our Customer Service Department if your control values fall outside the expected range.

Technical Assistance

For further information and technical assistance, contact the Customer Service Department 1-800-869-3538.



Manufactured by:



OraSure Technologies, Inc.



European Representative:

Qarad b.v.b.a
Volmolenheide 13
B-2400 Mol, Belgium



1745 Eaton Avenue • Bethlehem, PA 18018
610-882-1820 • 610-882-1830 Fax • 800-869-3538
www.orasure.com

rev. 05/08 10129